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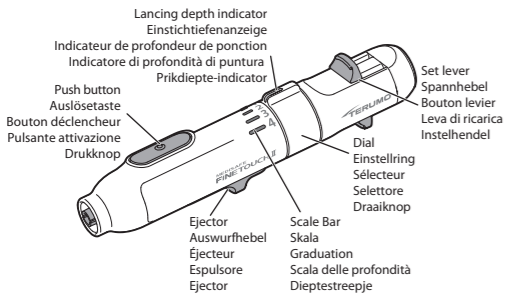
Fig. 6 / Abb. 6 / Fig. 6 / Fig. 6 / Afb. 6

ENGLISH

Please read carefully before use. Consult with your doctor, nurse or pharmacist if you need any advice or information on the device. If you have difficulties or if you think that the device may have led to an undesirable side-effect.

Intended Purpose
MEDISAFE FINETOUCH II Lancing Device is intended for capillary blood collection from the fingertip, earlobe or heel using a single-use FINETOUCH Lancet.

Indications
MEDISAFE FINETOUCH II Lancing Device is indicated for use when capillary blood sample for measuring blood glucose is required.

Contra-indications
None

Patient target group
Persons who require capillary blood testing.

Intended user
MEDISAFE FINETOUCH II Lancing Device should be used by a healthcare professional or under the order of a physician by a lay person.

Clinical benefit
MEDISAFE FINETOUCH II Lancing Device has an indirect clinical benefit - indirect medical effect since it is used to collect capillary blood sample for measuring blood glucose.

Caution

- Do not disassemble the MEDISAFE FINETOUCH II Lancing Device.
- If the MEDISAFE FINETOUCH II Lancing Device is deformed or damaged, especially the tip part where the lancet is inserted, stop using and replace the MEDISAFE FINETOUCH II Lancing Device.
- Before pricking, keep selected puncture site clean or disinfect by using an alcohol swab.
- Make sure that blood or devices contaminated with blood, including tissue paper and other items you have used, do not come into contact with any other person.
- Keep away from children.
- Do not store the MEDISAFE FINETOUCH II Lancing Device with the lancet attached. It can cause malfunction of the lancet depth adjustment.

With regard to FINETOUCH Lancet, please be careful of the following precautions.

- For single use only. Do not reuse. Do not re-sterilize. Do not reprocess.
- Reprocessing may compromise the sterility, biocompatibility and functional integrity of the device.

Instruction for use

- Before pricking, keep selected puncture site clean or disinfect by using an alcohol swab.
- Turn the Dial to select the desired lancet depth. The MEDISAFE FINETOUCH II Lancing Device allows you to choose between 5 levels of lancet depth. (Fig.1)
 - Soft or thin skin (shallow lancet depth)..... ♥ 1
 - Normal skin 2, 3
 - Hard or thick skin (deep lancet depth) 4

Caution in Use

- Do not set the lancet depth indicator between two depth markers.
- Make sure that the lancet depth indicator is pointing at a scale bar on the dial. (Fig.2)
- Start with the shallow lancet depth to find the appropriate lancet depth.

Caution in Use

- Only use FINETOUCH Lancet.
- Do not use the lancet when the packaging is damaged, when the lancet cap is not in place, when the lancet is damaged or any defect is found.
- When you insert the lancet into the MEDISAFE FINETOUCH II Lancing Device, be careful about the direction of the lancet. Insert it straight.
- Insert the lancet as far as it will go until you hear a clear "Click"; make sure not to press the Push button. (Fig.4)
- If the Push button is pressed accidentally, keep the lancet attached and pull the set lever until you hear "Click" (or MEDISAFE FINETOUCH II Lancing Device with set lever only, Fig. 5) or replace the lancet with a new one.
- Once the lancet is detached from the MEDISAFE FINETOUCH II Lancing Device, do not reinsert it. If you reinsert the lancet, the needle will remain exposed without the "Click". Replace detached lancet with a new one.

Caution in Use

- The needle may come out from the tip of the lancet if there is a strong impact on the lancet before disposal.
- If product fails to operate correctly, please stop using and replace the MEDISAFE FINETOUCH II Lancing Device.

Cleaning instruction

Clean the MEDISAFE FINETOUCH II Lancing Device as required using a mild liquid cleaner or a soft tissue moistened with water. In addition, the MEDISAFE FINETOUCH II Lancing Device can be disinfected by wiping the surface of the device with ethanol, isopropyl alcohol or chlorhexidine gluconate according to the instructions of the disinfectants manufacturer. Do not immerse the MEDISAFE FINETOUCH II Lancing Device completely in liquid.

Storage conditions

- Avoid exposure to water and direct sunlight.
- Do not store at extreme temperature and humidity.

Reference period for use

Around 10,000 times in normal use (In-house test).

Package

1 unit/ box

Incidence reporting

If during the use of this device or as a result of its use, a serious incident has occurred please report it to the manufacturer and/or its authorized representative and to your national authority.

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