NO COPY AREA

NO COPY GLUE AREA

3M Cavilon™ Durable Barrier Cream

27.4375 x 10.625 to 4.00 x 1.375

3M Cavilon™ Durable Barrier Cream

NO COPY GLUE AREA

NO COPY AREA

3M™ Cavilon™ Durable Barrier Cream

Product Description
3M™ Cavilon™ Durable Barrier Cream is a pH balanced, hypoallergenic, concentrated cream that provides skin protection from body fluids.

Indications for Use
Protection of at-risk skin from damage associated with incontinence and related symptoms such as anal itch. It may also be used to moisturise and condition dry skin.

Ingredients:
Water (Aqua), Dimethylol Adipate, Acrylate Terpolymer, Cocos Nucifera (Coconut) Oil, PPG-15 Stearyl Ether, Glycerin, Isopropyl Palmitate, White Mineral Oil (Paraffinum Liquidum), Trimethylsilyloxyoctylate, Paraffin, Dimethicone, Phenoxethanol, Benzoic Acid, Dihydroxyacetone, Magnesium Sulfate.

Warnings
FOR EXTERNAL USE ONLY. Avoid contact with eyes. If condition worsens or does not improve within 7 days, consult a physician.

Directions for Use
Gently cleanse skin as required. Apply Cavilon Durable Barrier Cream sparingly to cover entire affected area.

Precautions
Cavilon Durable Barrier Cream may increase the adherence of some adhesive products.

Storage/ Shelf Life/Disposal
For best results, store at room temperature. Do not freeze. Avoid excessive heat and humidity.

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Explication of Symbols
Symbol Title Symbol Description and Reference
Authorized Representative in European Community EC REP Indicates the authorized representative in the European Community.

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solventum Internal Production Team
Solventum Production Artist: Izabela Bassa
This artwork has been produced by the Solventum Internal Production Team
Date: 2025-14-03
Artwork ID: 0233475.2
Material Spec #: 34-2001-8512-1
Requestor: Katarzyna Zacharczykova
PID #: 20232\_s1

FOR PRINTER USE ONLY: This barcode symbol was produced using a bar width adjustment of .0007". If this does not meet the needs of the printing process or equipment to be used, the printer is responsible to increase the file using the appropriate bar width adjustment while meeting Solventum quality requirements.

Black Process Ink

PROOF REQUIRED