Evolsin® Wärmegürtel

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Evolsin® Heat belt

 **Please read carefully the information leaflet before use.**

- The **Evolsin® heat belt** provides soothing heat for back pain. The heat belt is made from a soft, skin-friendly material.
- DO NOT USE**
- on damaged or diseased skin
 - if there is swelling, bruising or inflammation
 - if the heat pads are damaged.
 - if the expiry date has been exceeded.
- Consult a doctor before use if you suffer from diabetes, poor circulation, a heart condition or rheumatoidarthritis, when taking medication, or if you are pregnant.

- DIRECTIONS FOR USE**
- Tear open the sachet containing the heat pad (do not cut), and remove the pad.
 - The heat pad is activated by the air. Do not open until it is to be used
 - Place a heat pad in each of the specially designed pockets.
 - Do not put more than one heat pad in each pocket. Ensure that the pads are flat and not twisted or doubled over as this increases the risk of burns.
 - Wrap the heat belt carefully around the body and secure it with the hook-and-loop closure. Do not apply any pressure by wearing a belt or tight-fitting clothing over the heat belt. Allow air to circulate around the heat belt.
 - The heat pads warm up slowly, and after approximately 30 minutes generate their soothing,therapeutic heat that can last for up to twelve hours.
 - After use remove the heat pads from the belt and dispose of in household waste.
 - Each pad can be used only once; the heat belt is reusable.
 - The heat belt is only designed for external use in the lumbar or hip region.

- SAFETY INFORMATION**
- Only use under supervision with children, older people over the age of 55 years or the disabled.

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- If used incorrectly, the heat pads can cause burns to the skin, even at low temperatures.

- Please follow the instructions for use carefully.**
- In rare cases, heat-generating products can cause burns!
 - People with sensitive skin or anyone who has previously experienced allergic reactions to medicines or cosmetics should wear the heat belt over thin clothing and not directly on the skin.
 - Do not use in combination with pain relief cream, medicinal lotions, creams, ointments or together with hairspray, deodorant or perfume.
 - Do not wear over a tattoo.
 - Never heat in a microwave!
 - If the heat pads get uncomfortably hot or cause a skin rash, remove them immediately. In case of skin irritation, or if it feels too hot or unpleasant, remove the belt with the pads immediately. Use of the belt can cause burning. Check your skin regularly during use.
 - Please do not lie or sit on the heat belt. Do not lean against hard surfaces!
 - Do not use immediately after a bath or shower.
 - Do not use when sleeping.
 - To avoid an increased risk of burns, keep the product away from any MRT (magnetic resonancetomography) equipment.
 - Depending on skin type, allow at least 48 hours before reuse.

- STORAGE**
- The heat pad should be stored at room temperature (25° C).
 - Store heat pads out of the reach of children and pets.
 - In case of accidental swallowing, induce vomiting and seek medical assistance immediately.
 - If the contents of the heat pad comes into contact with the skin or eyes, rinse thoroughly with water immediately.
 - Protect from humidity.

- INGREDIENTS:**
- Iron powder, activated carbon, wood flour, salt, water.

Please follow the instructions for using this heat pad carefully as the Evolsin Medical UG company can assume no responsibility in case of improper use.

Retain the instructions for use until the heatpatches have all been used. If you plan or intend to use the heat pad for medicinal purposes, please consult a doctor before use.

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**Hersteller / Manufacturer / Fabricant / Produttore / Fabricante:** Koolcare Technology Co., Ltd.
Prolinx GmbH, Brehmstraße 56, 40239 Düsseldorf, Germany info@eulinx.eu
MedPath Limited, 27 Old Gloucester Street, London, United Kingdom, WC1N 3AX, Tel: +44 776 280 3783