Scan for more information

**Advancis**
MedicalAdvancis Medical, Lowmoor Business Park,
Kirkby-in-Ashfield, Nottingham, NG17 7JZ, United KingdomTel: +44 (0) 1623 751500
Fax: +44 (0) 871 264 8238
Email: info@advancis.co.uk
Web: www.advancis.co.uk**Advazorb Border®/Advazorb Border Lite®** **KO**

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Advazorb Border®**Advazorb Border Lite®****Hydrophilic foam dressing with soft silicone wound contact layer and border****Instructions for use****Advancis**
Medical

Advazorb Border®/Advazorb Border Lite® **EN**
Hydrophilic foam dressing with soft silicone wound contact layer and border

Product description

A self adherent absorbent dressing with a Silfix® soft silicone wound contact layer and border. The dressing has a hydrophilic foam absorbent pad with high fluid handling capability. The soft silicone will adhere to surrounding dry skin but not to a wet wound. This is an atraumatic dressing which means it is designed to minimise the pain and trauma associated with dressing change. The film backing is breathable and prevents strike through.

Indications

A wide range of exuding wounds including: Leg, foot and pressure ulcers, traumatic wounds, surgical wounds, abrasions and blisters.

Contra-indications

Arterial bleeds and heavily bleeding wounds. Do not use if allergic to silicone.

Directions for use

Remove the clear plastic liners and place directly onto the wound ensuring the absorbent pad covers the entire wound area with an overlap of 1-2cm. Can be used under compression.

Frequency of change

The dressing can be left in place for up to 7 days but should be changed when exudate is visible in the centre pad or the pad swells due to absorbed exudate.

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